

**RiJe® Petite Cup Instruction for Use**  
*Model RJP*  
[www.rije.ca](http://www.rije.ca)

**Indications**

For people suffering with difficulty in swallowing (dysphagia), developmental disabilities, motor impairment, or requiring predetermined doses of liquid.


**Precautions and Warnings**

- Please seek the advice of a medical professional prior to using this cup.
- Instructions must be read prior to use.
- **CHOKING HAZARD. Small Parts. Adult Supervision Required.**
- Wash the cup prior to use.
- RiJe® Petite Cup is designed to be used with thin and slightly thick liquids (IDDSI Standards 0 and 1). Carbonated beverages will render RiJe® Petite Cup ineffective for delivering an accurate dosage.
- Do not use RiJe® Petite Cup for boiling liquids in a microwave.
- Hold RiJe® Petit Cup upside down when opening to prevent spillage.

**Features**

- Dispenses controlled boluses of thin or slightly thick liquids;
- Allows pre-setting the bolus volume from 4ml to 12ml. The volume range outlined in the table below;
- Holds up to 150 ml of liquid;
- Allows drinking the controlled liquid boluses from the cup's rim or through a regular straw.
- Good nose clearance;
- Ergonomic handles provide secure grip;
- Spill-control does not allow spilling more than one bolus of liquid if the cup is dropped;
- Stable base;
- Dishwasher safe;
- BPA Free.

**PARTS**

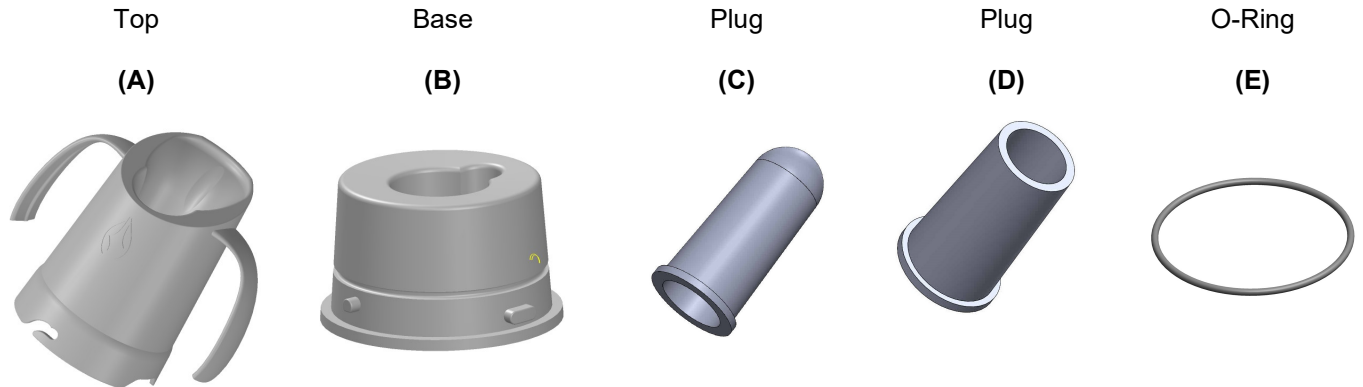

**Choosing correct Plug for Thin Liquids (IDDSI Standard 0) or Slightly Thick Liquids (IDDSI Standard 1).  
 Range of Dispensed Volumes.**

**Note:**


- The Plug should be chosen according to the viscosity (thickness) of a liquid being used at the moment.
- **The viscosity (thickness) of a slightly thick liquid is highly dependent on temperature of the liquid, the type of thickening product (if any) is being used, and the time it was allowed to thicken.**

| Liquid's Viscosity                                      | Dispensed Volumes |                  |                                    |
|---|-------------------|------------------|------------------------------------|
|   | Using Plug C      | Using Plug D     | Using RiJe Petite Cup without Plug |
| Thin Liquids Cold or Hot.<br>IDDSI Level 0 Thin Liquids | Dispenses 5 ml    | Dispenses 8 ml   | Dispenses 12 ml                    |
| Slightly Thick Liquids<br>IDDSI Level 1 Slightly Thick  | NOT TO USE        | Dispenses 4-5 ml | Dispenses 8 ml                     |

## Assembly

### Step 1:

Insert desirable Plug firmly by **PRESSING DOWN**. Use either **C** or **D** depending on the thickness of the liquid used at the moment (please see table above).

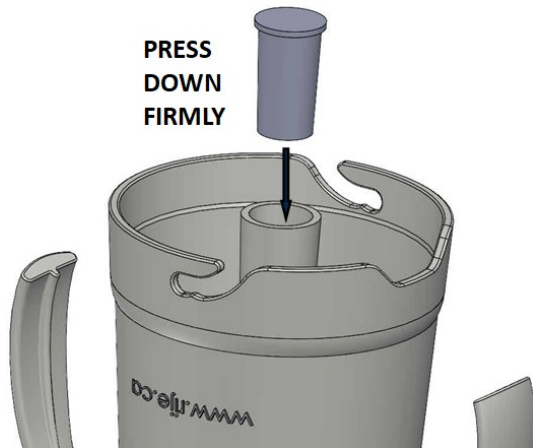


### Step 2:

Fill liquid **ONLY** into Upper Cap A.




### Step 3:

Put Base **B** over Upper Cap **A**. Make sure that the sealing O-ring sits in the groove of Base **B**.

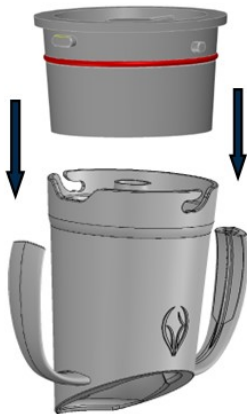

### Step 4:

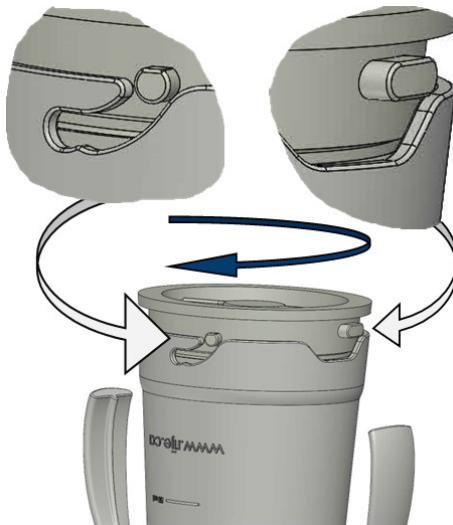


To lock the cup, turn Base **B** against Upper Cap **A**. Direct each knob to its corresponding slit as shown below. Turn tight until the knobs **reach to the end of their slits**.



### Step 5:

The cup delivers liquids in an upright position. **Put the cup upright after each sip** for 2-3 seconds to allow next portion to be delivered.

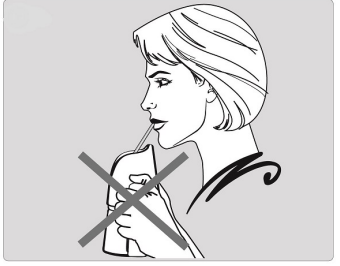

| Correct drinking position.  | Incorrect drinking position  |   |
|---|--|---|
| Drinking controlled volumes from an open cup.                                       | When drinking controlled volumes using a regular straw, always slightly incline the cup to ensure accurate dosage. | When using a straw slightly incline the cup to ensure accurate dosage.                |
|  |                                 |  |